

Using your Nylon Ball Strap

Attaching the Strap to the Temple

- The two ends of the chum strap have adjustable loops that are used to secure the chum to the temple of the glasses
- Slide the loop up the temple as shown in Figure 1, then push the ring up towards the temple to tighten the loop.
- The final position of the strap should look similar to Figure 2. Repeat for other side

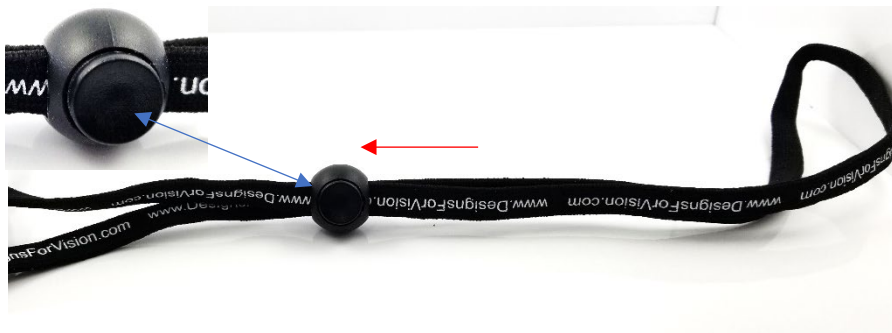


Figure 1



Figure 2

Figure 3



Tightening the Strap Around the Head

- Once the strap is attached to the glasses, place the glasses on.
- Push the button in and slide the bead towards the head to tighten the strap as seen in Figure 3
- To remove the chum strap, push the button in and push the beads back towards the back of the head

Recommended Cleaners:

- Clorox Verasure Alcohol/Bleach Free, Caviwipes, Lysol IC(if available), Sono Disinfectant Wipes, Opti-Cide Medical Wipes
- Do not spray onto product, spray cleaner onto a cloth and wipe down
- Cleaners should be non-porous/porous plastic safe or all surface safe