

Figure 1



Figure 2

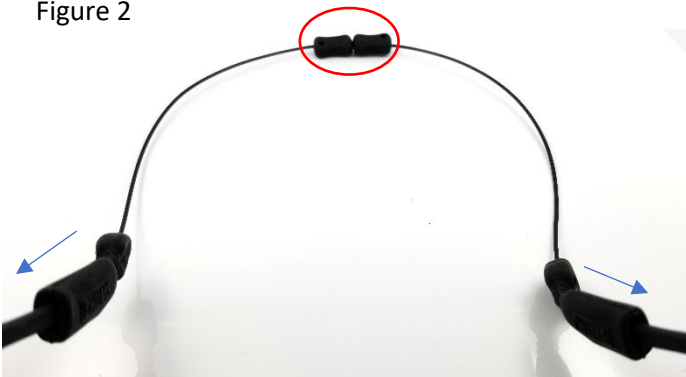


Figure 3



Using your Chums® Adjustable Orbiter Strap

Attaching the Orbiter Strap to the Temple

- The two ends of the strap have openings in them to slide over the end of the temples seen in Figure 1.
- Slide them up the temple until they resemble the position shown in Figure 2

Tightening the Orbiter Strap

- Once the strap is attached to the glasses, place the glasses on. Grab the two beads on the back of the strap, circled in Figure 2.
- Pull them apart to tighten the strap snug to your head as shown in Figure 3
- To remove the chum strap, push the beads back towards the back of the head in the direction of the red arrows

Recommended Cleaners:

- Clorox Verasure Alcohol/Bleach Free, Caviwipes, Lysol IC(if available), Sono Disinfectant Wipes, Opti-Cide Medical Wipes
- Do not spray onto product, spray cleaner onto a cloth and wipe down